

# System3

## Instrument wetting/pre-cleansing agent

To aid instrument reprocessing during Decontamination Holding Time (DHT).

Use on soiled instruments when a precleaning or predisinfection is not possible in the service or if the manual or mechanical cleaning area is outsourced.

### System3 for a TRIPLE action

Maintains soil moisture and delays instrument drying.

Active agents aid reprocessing.

Prevents rusting and pitting.

Foaming formula for an uniform coverage of the instruments.

#### Directions for use

1. Spray SYSTEM 3 on the bottom of a suitable container. 2. Immediately following use, place soiled instruments in the container. 3. Spray directly onto instruments ensuring complete coverage. 4. Keep the container covered (instruments should be processed within 24 hrs following application). 5. Prior to final decontamination, rinse thoroughly under running water.

#### Compatibility

Stainless steel, Polycarbonate, Polyethylene, Polypropylene.

#### Composition

Liquid preparation. Anionic surfactants, enzyme.

#### Cautionary notes

Read label before use.

#### Characteristics

- Physical state : liquid
- pH : 9-9,5
- Fragrance : free
- Color : colorless



#### Presentation

800mL spray bottle

